


NEW

QuikClot[®] **COMBAT GAUZE**[™]

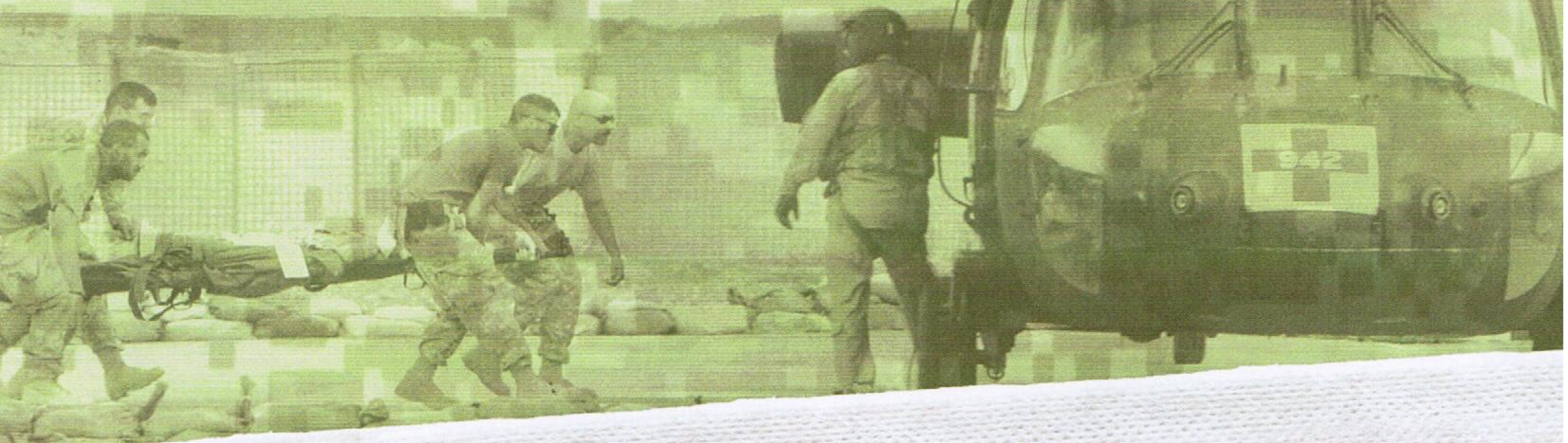

**PACK IT. WRAP IT.
STOP BLEEDING COLD.**

- **Pliable: Fits any size or shape wound.**
- **No heat.**
- **Easy to pack.**
- **Easy to remove.**



"QuikClot" is a registered trademark and
"QuikClot Combat Gauze" is a trademark of Z-Medica Corporation.

Z-MEDICA CORPORATION 4 Fairfield Blvd., Wallingford, CT 06492
p: 203-294-0000 f: 203-294-0688 www.z-medica.com info@zmedica.com

NEW


QuikClot® COMBAT GAUZE™

The ease of gauze. The power of QuikClot®.


New QuikClot® Combat Gauze™: The first gauze to stop arterial and venous bleeding in seconds. Powerful yet cool, so it won't burn. Pliable to pack into even penetrating traumatic wounds. Flexible to cover any size or shape. No mixing. No measuring. No heat. Easy to use. Easy to remove. New QuikClot® Combat Gauze™



Actual size: 3 inches x 4 yards.

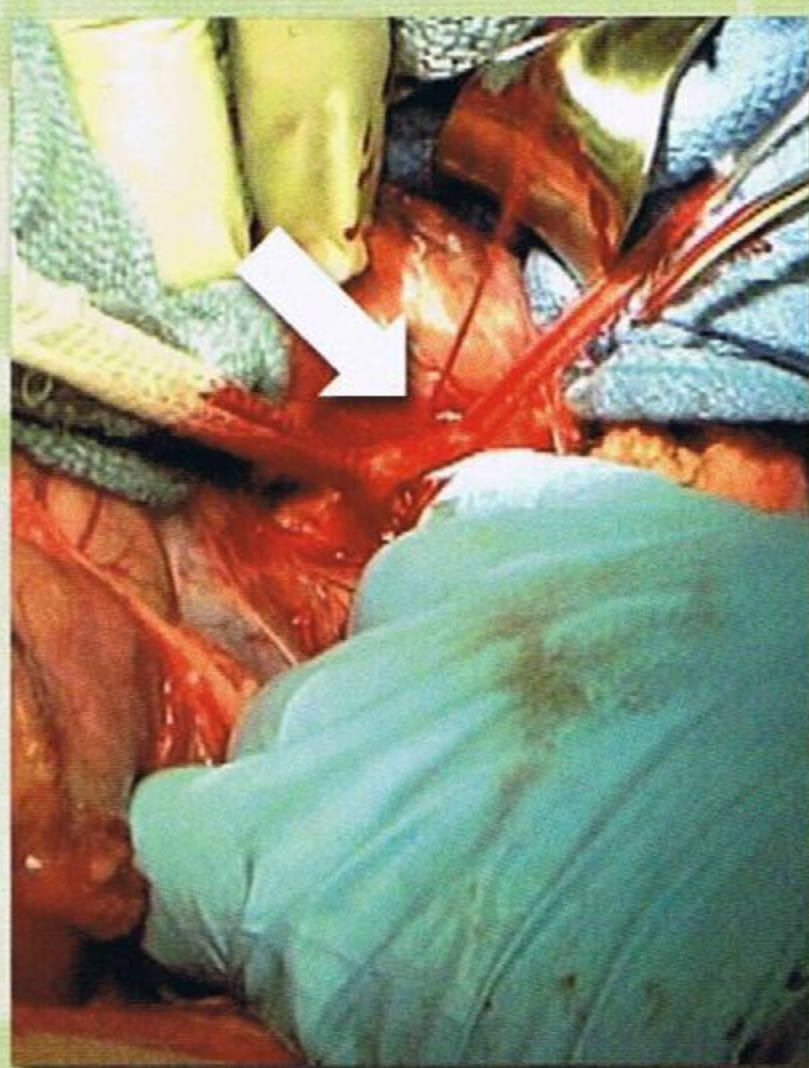


Aorta first puncture, arterial bleeding under pressure (arrow).

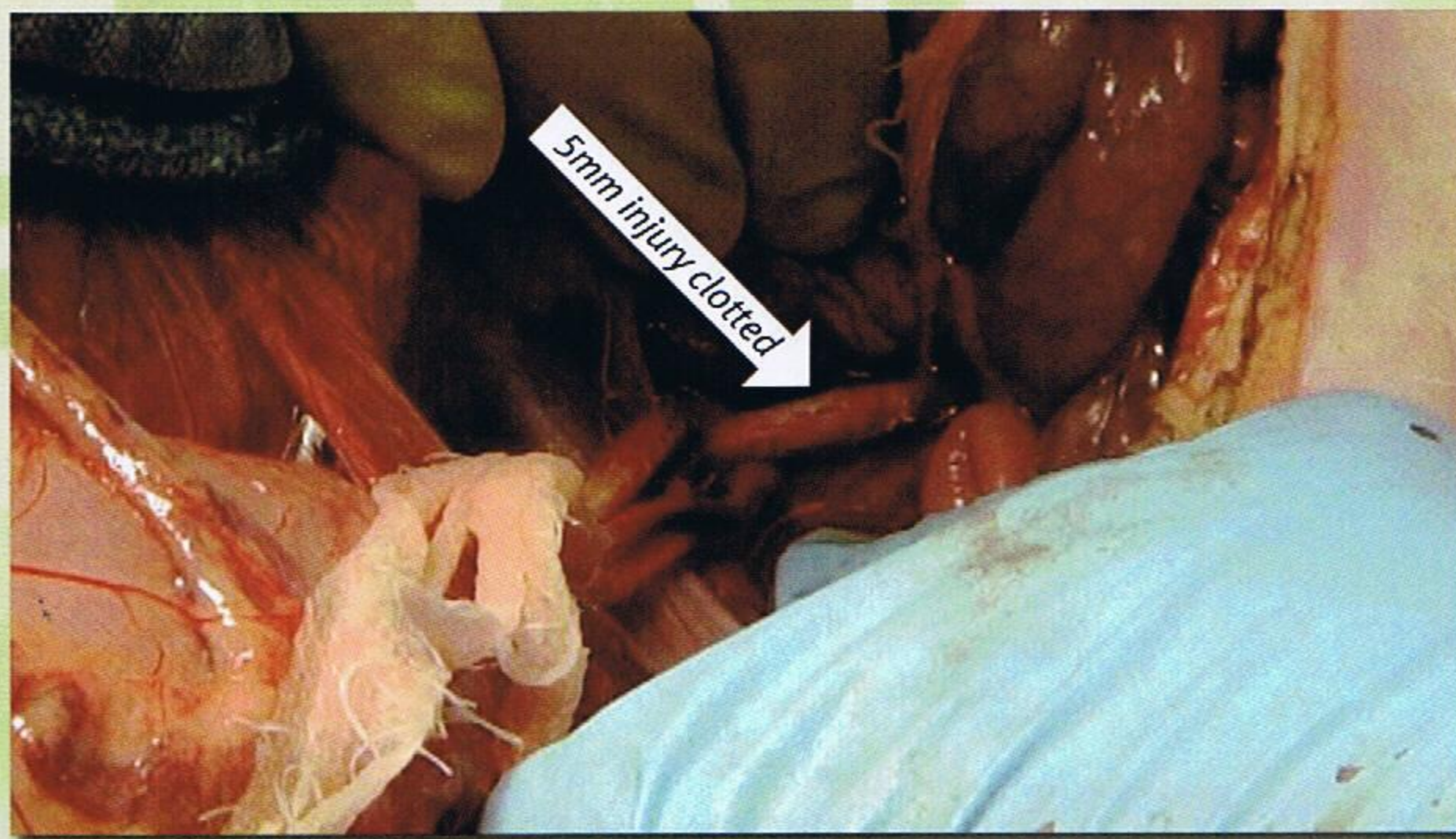


Aorta 30 minutes following injury and QuikClot® Combat Gauze™ application and subsequent removal.

DIRECTIONS FOR USE

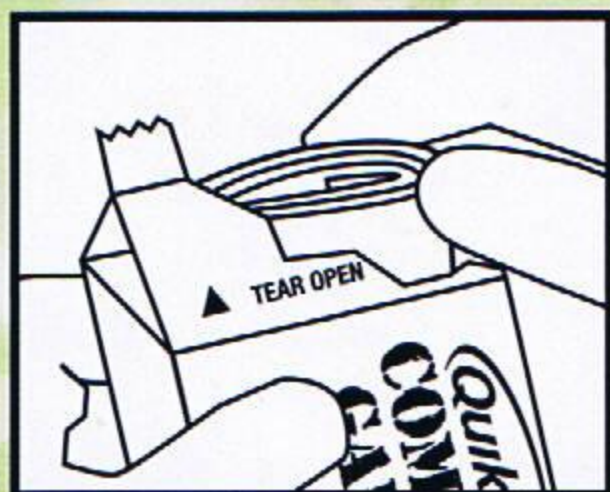


1. Open package and remove **Combat Gauze**. Keep the empty package.

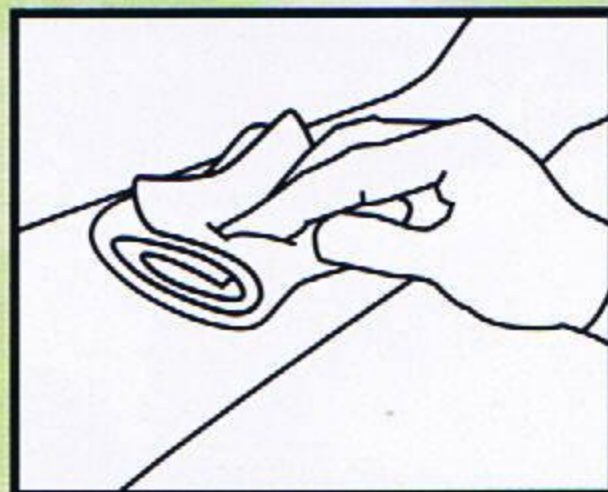

2. Place **Combat Gauze** over wound and use it to apply pressure directly over bleeding source. (More than one **Combat Gauze** may be required).



3. Continue to apply pressure for 3 minutes or until bleeding stops.

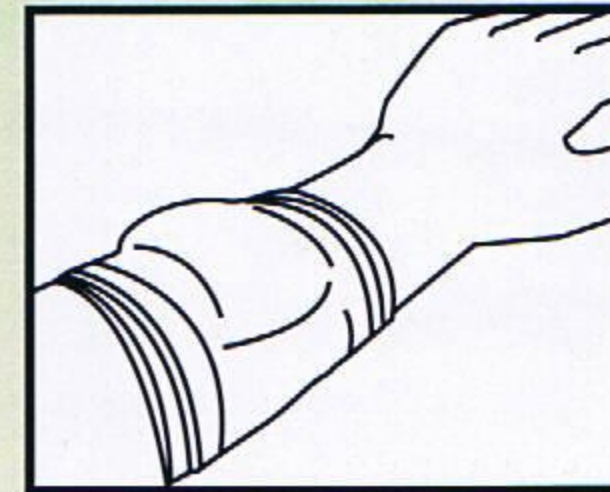

4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes.

© 2007 Z-Medica Corporation. "QuikClot" is a registered trademark and "QuikClot Combat Gauze" is a trademark of Z-Medica Corporation.

Z-MEDICA CORPORATION 4 Fairfield Blvd., Wallingford, CT 06492 p: 203-294-0000 f: 203-294-0688 www.z-medica.com info@zmedica.com